

IMPORTANT

BEFORE UNBOXING

LIFT THE BACK AND SEAT OUT OF
THE CARTON

STILL WRAPPED TOGETHER

THEY ARE CONNECTED BY A BRAKE CABLE
THAT MAY BE DAMAGED IF THE BACK IS
REMOVED SEPARATELY



Dal Slide n Dine utility chair.

ASSEMBLY INSTRUCTIONS




1

Fit backrest to base

2



Push backrest firmly onto support posts and fit screws. Some downward pressure may be required on the back for the screw holes to line up.

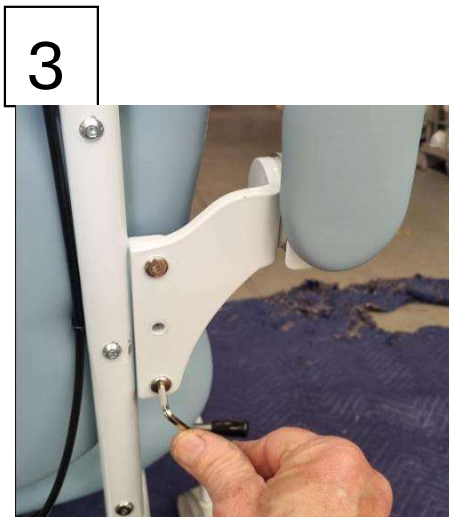

3

Fit Armrests to frame in the middle position, using only 2 screws.

The middle hole is only used when fitting the arms at the higher or lower position

4



When all screws are in, tuck the brake cable out of the way.

Finished